



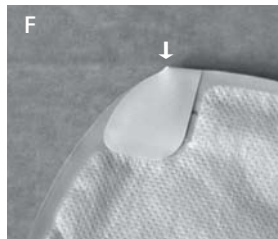
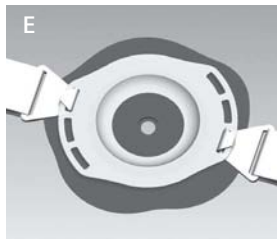
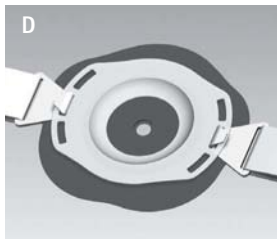
Flexima® / Softima®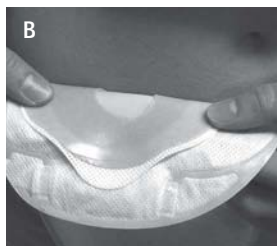




- EN** Drainable pouch
Instructions for use
- ES** Bolsa abierta de ostomía
Instrucciones de uso
- FR** Poche vidable
Mode d'emploi
- DE** Ileostomiebeutel
Gebrauchsanweisung
- IT** Sacche per ileostomia
Istruzioni per l'uso
- NL** Ileo opvangzakje
Gebruiksaanwijzing
- DA** Tømbar pose
Brugsvejledning
- BG** Торбичка с оттичане
Инструкция за употреба
- CS** Výpustné sáčky
Návod k použití
- FI** Tyhjennettävä sidos
Käyttöohje
- HU** Egyrészes nyitott zacskó
Használati utasítás
- JA** 開放型パウチ
取扱説明書
- PL** Jednoczęściowe otwarte worki stomijne
Instrukcja użycia
- PT** Bolsa drenável
Instruções de utilização
- RO** Sac drenabil
Instrucțiuni de utilizare
- RU** Открытый калоприемник
Инструкция по применению
- SK** Výpustné vrecko
Návod na použitie
- SV** Tömbar påse
Bruksanvisning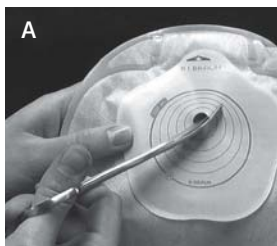

Roll'Up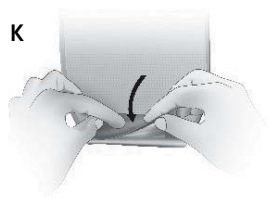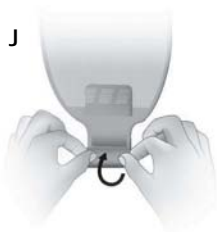

Flow Control

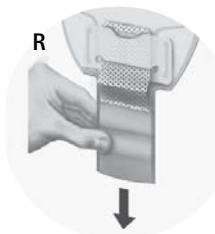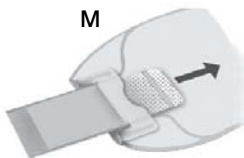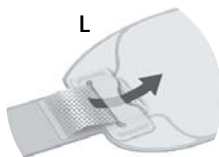EN Instructions for use

One-piece drainable pouch with flat or convex skin protector*, for collection of body effluents for any person having undergone an intestinal surgery of the ileostomy type.

1 Preparation of the skin:

- Wash your hands.
- Gently clean the area around the stoma with lukewarm water and carefully dab with soft tissues.
- Avoid using any disinfectant (ether, alcohol or other antiseptic...) as it damages the skin and prevents the proper adhesion of the pouch.

2 Fitting your appliance:

- Cut the skin protector so it fits the size of your stoma, taking good care not to perforate the film of the pouch (**do not cut beyond the limit**) (A).
- Close the outlet system. Separate the pouch walls by crumpling it slightly to introduce air into it.
- Peel the protective paper from the skin protector.
- Position the bottom of the flange opening around the base of the stoma (B).
- Press the bottom part of the skin protector smoothly onto the skin, then place and press onto the upper part.
- Smooth the entire surface of the skin protector, working from bottom to top, to ensure perfect adhesion (C).

If you wish to use a belt (with a convex product), choose two notches that are diametrically opposed (D-E).

3 Removing your appliance :

- Pull slightly on the top of the pouch and remove the pouch from the top to the bottom, while maintaining gentle pressure on the skin around the stoma with your other hand.

4 Use of patch cover for filter protection:

If you wish to inactivate the filter (bath, shower...), stick an adhesive patch on both sides of the top of the pouch so that the hole for gas evacuation is blocked (F).

5 Drainable pouches:

Drainable pouch with a classical clamp*

- Please ensure that your pouch is closed before applying it.
- Use the clamp provided in the box to close the pouch (G-H-I).
- When you want to empty the pouch, remove the clamp and direct the open end of the pouch over the toilet. Wipe the end of the pouch with a tissue after it has been emptied and reapply the clamp.

Drainable pouch with the Roll'Up integrated outlet*

- Closing of the outlet:

- Fold three times the remaining sleeve onto itself (J).

-
- Press all the length of the self-grabbing strips to seal the pouch (K).
- For more comfort, fold up the tongue under the soft backing material.

- Emptying of the pouch:

- Position yourself to empty your pouch as normal and free the evacuation sleeve by lifting the tab up. Caution: emptying starts at this stage.
- Expel the remaining matter in the sleeve by pressing it slightly from top to bottom. Clean carefully the open end of the sleeve with a non-fluffy material.
- Then, close the pouch.

Drainable pouch with the Flow Control integrated outlet, allowing a controlled release of stools*

- Close the flow control outlet according to the provided instructions for use (L-M-N-O).
- To empty your pouch, lift the tongue and direct the opening of the pouch towards the toilet without pulling the evacuation sleeve (P-Q).
- Free the stool flow by pulling the evacuation sleeve (R).
- Then expel the remaining matter in the sleeve by pressing it slightly from top to bottom.
- Pull up the tongue as high as possible to stop the flow and clean the end of the evacuation sleeve with a nonfluffy material.
- Finally, close the system.

Drainable pouch for high flow*

- Close the pouch with the plug included in the box
- To empty your pouch, remove the plug and direct the opening of the pouch towards the toilet. Then, clean the end of the evacuation outlet with a non-fluffy material and close with the plug.
- If it's necessary, the pouch can be connected to a higher capacity collecting pouch (Flow Collector).

**Availability may vary from one country to another.*

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B. Braun Medical B.V.
B. Braun Medical AB